



Perio Protect Digital Lab

3D Tips and Tricks

3Shape / Trios

Add Perio Protect Lab as a preferred lab:

- 1) Login to the 3Shape website.
- 2) Click on **CONNECTIONS** in the navigation bar at top of screen.
- 3) In the **CONNECT TO A PRACTICE or LAB** window, type our email: **3D@Perio-Protect.com**.

Once we accept, you can start sending cases. Remember to include Rx and periodontal charting in the case details.

Medit


Go to Medit's website to add Perio Protect Lab as a partner:

- 1) Click on **PARTNERS** in the navigation bar on the left.
- 2) Under **ADD NEW PARTNERS**, in the **SEARCH BY CLINIC NAME, EMAIL ADDRESS** window, type our email: **3D@Perio-Protect.com**.

If Medit does not find us, click on the blue **INVITE PARTNER** button on the right and enter the same email.

CareStream / CS Connect

Invite Perio Protect Lab to be a laboratory partner:

- 1) Login to CS Connect as a dentist.
- 2) In the tool bar, click this icon: 
- 3) In the **PARTNERS** window, click **INVITE A PARTNER**.
- 4) In the email field enter: **3D@Perio-Protect.com**.
- 5) Click **SEND**.

Once we accept, you can start sending us cases.

Planmeca

- 1) Export 3D scan from Planmeca and save as 3Shape (preferred) or STL file.
- 2) Go to: **www.PerioProtect.com/3D**.
- 3) Click **UPLOAD**, then click **CHOOSE FILES**.
- 4) Select file, enter doctor's full name and office email.
- 5) Click **+ADD MORE FILES** to upload Rx and perio charting.
- 6) Click **UPLOAD**.

iTero

Call iTero at **800-577-8767** (ext 1 or 2) to add Perio Protect Lab as a preferred lab. Or look us up in iTero by our ID: **38291**.

Sirona Cerec

Add Perio Protect, LLC as a Favorite Lab in your system:

- 1) In the SIRONA Connect heading, go to **MY FAVORITE LABORATORIES**.
- 2) Click the **SEARCH** option.
- 3) Enter zip code: **63125**.
- 4) Select **Perio Protect, LLC**.

Other scanning software

Upload STL files directly through our website.

- 1) Go to: **www.PerioProtect.com/3D**
- 2) Click **UPLOAD**.
- 3) Click **CHOOSE FILES**.
- 4) Select file, enter doctor's full name and office email.
- 5) Then click **UPLOAD**.

This method is HIPAA compliant and allows you to send STL files, Rx, and perio charting at the same time.

Send Rx & periodontal charting!

DON'T FORGET TO SEND the Rx and PERIO CHARTING.

You have **3 options**:

- Fax both to **888-511-2308**.
- Upload the files directly to our website.
Go to: **www.PerioProtect.com/3D**. Click **UPLOAD**, then click **CHOOSE FILES**. Select files, enter doctor's full name and office email. Then click **UPLOAD**.
- Email both to **3D@PerioProtect.com**.

Tips and file preferences

- 3D scans must be in STL format.
- Models must be solid, with a low-profile base.
- File per arch, with arches oriented in occlusion.

Questions? Call 314-474-0467

Perio Protect Laboratory
9701 Green Park Industrial Drive
St. Louis, MO 63123

Lab Phone: 314-474-0467
Fax: 888-511-2308
www.PerioProtect.com/3D